Maxi Sky 440




Hand control



- ↑ Up button
- **Down button**

Control panel

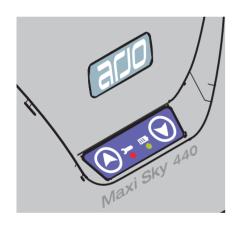



Up and down buttons



The up and down buttons are used to complete a transfer if a problem occurs with the hand control during the operation.



Maintenance required - Red light,

Flashing (no beep) -Maintenance required by your local agent.

Solid (with beep) -Maintenance is overdue. Stop using the lift and contact your local agent for service.



Battery charge level - Yellow light

Flashing - Batteries are charging. Solid - Batteries are charged.



Automatic emergency brake




Risk of falling

The emergency brake is an additional safety feature that automatically prevents the patient from falling in the unlikely event of a transmission or motor failure.

The emergency brake can only be used once. If it has been activated, call your Arjo agent to arrange for the unit to be replaced. The emergency lowering system cannot be used while the emergency brake is engaged. Use an alternative device to lower the patient.

Emergency shut-off

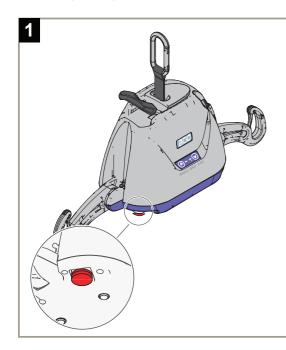


The operator can shut off the electrical power at any time by pressing the red mushroom-shaped emergency button under the lift.

The button is reset by twisting it clockwise in the direction of the arrows.


First time users should practice the shutoff manoeuvre before operating the lift with a patient.



Emergency lowering

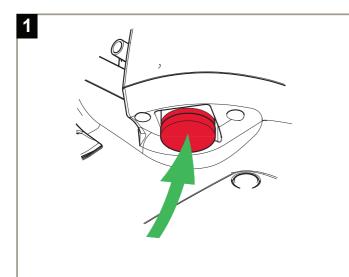


Press on the red emergency shut-off button under the lift. Move the lift with the patient over the bed or the wheelchair.

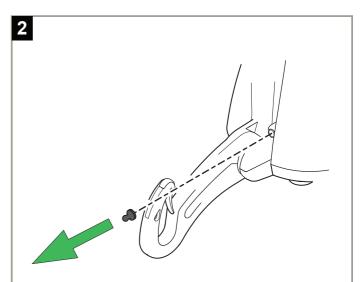

Remove the black plastic cap located on the

side of the lift.

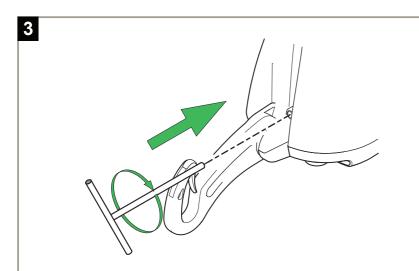

Insert the special tool into the opening. Turn it clockwise to lower the patient. The tool rotates the motor directly, so each turn of the tool will slightly lower the patient.

Once the patient has been lowered safely on a bed or into a wheelchair, remove the tool from the lift. Call a qualified technician to service the lift.