

Maxi Sky 440



Clean and disinfect before new patients and when suspected to be contaminated.

Clean

1. Use a cloth soaked with water to remove excessive residues from the lift and its accessories from top to bottom.
2. Use a damp cloth with warm water mixed with a mild detergent.
3. Scrub the equipment from top to bottom, removing all visible residues.
4. Rinse with clean water any remaining residual detergent.
5. Wipe clean with a dry cloth.

CAUTION

Do not splash, drench or immerse the unit in water. The lift is an electrical device and water could damage your unit.

To ensure a better rolling surface for the trolley wheels, clean the inside of the rail every 4 months. To do so, insert a damp cloth in the opening and slide it from one end of the rail to the other.

Areas listed are most likely to contain dirt and microbes, pay attention to:

- Plastic enclosure junction
- Handset pocket
- Plastic cavities
- Enclosure corners
- Product label edge
- Rubber stand-offs
- Membrane edge
- E-stop cavity
- Screw cavity
- Enclosure junction
- Strap opening

Disinfect

6. Wipe the Maxi Sky 440 and accessories using a solution containing one of the compatible disinfectants.
7. Rub the lift vigorously when using the wipes, to promote an effective disinfection of its entire surface.
8. Do not use phenol, chlorine or any other type of solvent that may damage the finish.

