



USER MANUAL



WE HAVE YOU COVERED

Our passionate team brings you innovative, quality mobility solutions. Our products are reliable, sleek, and designed with simplicity so that you can rest assured that they work when you need them.

Thanks to our out of the box thinking, we are vigilant to meet your mobility needs with our simple and practical solutions. We recognize the important role that safety plays in consumer choices and are proud to offer a line of products that maintain the safety features found in your vehicle. Backed by third party verification for compliance to applicable Federal Motor Vehicle Safety Standards and Canadian Motor Vehicle Safety Standards, our products are engineered for safety and durability.

Our commitment to quality and safety along with our genuine desire to meet your mobility needs makes for a product we are sure you will be proud to own.





POWER-PULL

USER GUIDE

Entering your vehicle

Your new **POWER-PULL** is designed to make it easier to get up and down the ramp of your lowered floor minivan.

To use the POWER-PULL to go up the ramp, please follow the next few steps:

- After you have opened the door, deploy the ramp.
- Roll up beside the ramp and pick up the **POWER-PULL** hooks. You may find it useful to temporarily attach the hooks to the side of your wheelchair for the next step.
- Position yourself in line, in front of the ramp, and place the **POWER-PULL** hooks onto the **solid frame** of your wheelchair; (ie: not onto the removable foot rests) one on either side. **NOTE:** The length of the straps has been adjusted by the equipment installer. In a side entry lowered floor minivan, it is normal that one strap attached to the **POWER PULL** hook be longer than the other. **Always** place the longer strap and hook onto the right hand side of your wheelchair. In a rear entry lowered floor minivan, the straps for the hooks should be the same length. Placing either hook on either side makes no difference in this application.
- Retrieve your remote control, press and hold the **IN** button until you are all the way into the vehicle. At anytime, during the ascent you may release the **IN** button and the **POWER-PULL** will hold you in place.
- Once you are all the way up the ramp and safely into the vehicle, press the **RELEASE** button and remove the **POWER-PULL** straps. Only remove the **POWER PULL** once there is no chance you will roll down the ramp.

FOR YOUR SAFETY

Your **POWER-PULL** is equipped with several safety features. If you were to accidentally hit the **RELEASE** button while you are still going up or down the ramp, the **POWER-PULL** cable will not release. The **POWER-PULL** is designed to only release once there is no tension or weight on the cable.

Your **POWER-PULL** is also equipped with a security feature that prevents it from pulling the wheelchair over obstacles. The electronic module inside the **POWER-PULL** monitors the voltage during your usage. If it spikes over 10 amps, the **POWER-PULL** will automatically shut off and allow you to remove the obstacle or straighten out your wheelchair, (for instance: if the obstacle were the side of the ramp) all the while keeping the tension on the cable so you do not roll back.

WARNING

THE **POWER-PULL** IS NOT A TIE DOWN FOR YOUR WHEELCHAIR AND SHOULD NOT BE LEFT ATTACHED TO THE WHEELCHAIR DURING TRANSPORTATION. PLEASE USE A PROPER DOCKING SYSTEM FOR YOUR WHEELCHAIR.

⚠ WARNING: This product can expose you to chemicals including benzene, which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information, go to www.P65Warnings.ca.gov.





USER GUIDE

Leaving your vehicle

To use your **POWER-PULL** to get out of your vehicle, follow the next few steps:

- Open the vehicle door and deploy the ramp. Position yourself to back down the ramp. For safety reasons, it is strongly recommended to **ALWAYS** back down the ramp.
- Place the **POWER-PULL** hooks onto the **solid frame** of your wheelchair; (ie: not onto the removable foot rests) one on either side. **NOTE:** The length of the straps has been adjusted by the equipment installer. In a side entry lowered floor minivan, it is normal that one strap attached to the **POWER PULL** hook be longer than the other. **Always** place the longer strap and hook onto the right hand side of your wheelchair. In a rear entry lowered floor minivan, the straps for the hooks should be the same length. Placing either hook on either side makes no difference in this application.
- Retrieve your remote control, press and hold the **OUT** button. **NOTE:** At any time, you may release the button and the **POWER-PULL** will hold you in place.
- Once you have completed your descent, press the **RELEASE** button and unhook the **POWER-PULL** straps from your wheelchair.
- For a rear entry lowered floor minivan, simply place the **POWER-PULL** hooks and straps somewhere that will be accessible upon your return. For a side entry lowered floor minivan, place the **POWER-PULL** hooks and straps onto the ramp to allow you to have access to the **POWER-PULL** when the ramp deploys upon your return to your vehicle. Use the hook receiver that has been installed onto your ramp.

Stow the ramp and close the door.

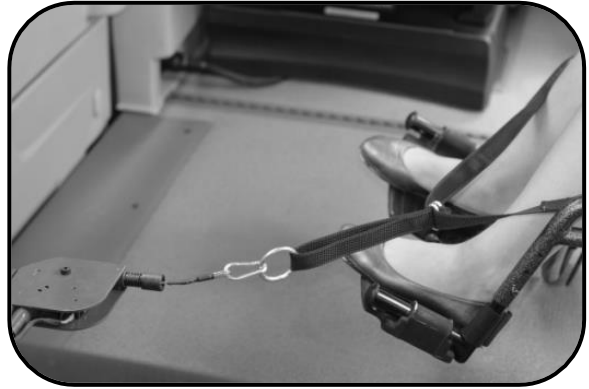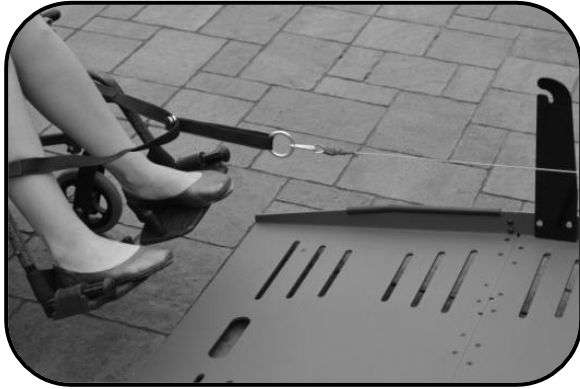

3 YEAR LIMITED WARRANTY POWER PULL

Adapt Solutions # 1 Ltd. (Adapt Solutions), warrants to the original purchaser of a **POWER-PULL** that the equipment is free from defects in material and workmanship for a period of three years from date of purchase.

During the first year of the warranty, Adapt Solutions will supply the replacement parts as well as a pre-set monetary amount (determined by Adapt Solutions) for the repair if a defect in material or workmanship is discovered. After the initial year of this warranty, only parts and components are covered. This warranty does not cover labour and other services after the initial year. Freight and other related repair charges will be the responsibility of the original purchaser.

The only remedy for a defect in one of Adapt Solutions products shall be the repair or the replacement, at the discretion of Adapt Solutions, of the defective part or component. If repair or replacement is not commercially practical or cannot be timely made, Adapt Solutions may decide to refund the purchase price of the equipment instead of repairing or replacing the original equipment.

In no event shall Adapt Solutions be responsible for indirect, incidental or consequential damages, whether such damages arise from claims based on contract, warranty, tort (including negligence), strict liability or product liability.

All implied warranties, including any warranty of merchantability or fitness for a particular purpose, are limited in their duration to the length of the warranty stated above for the affected component.

This warranty is to the original purchaser only, and excludes product damage due to installation error, product misuse, product abuse, accidents, physical damage, damage in shipment, modifications not made by Adapt Solutions, or repairs undertaken by anyone other than authorized distributors.

TO OBTAIN WARRANTY PARTS OR REIMBURSEMENT

You must follow these procedures:

1. Obtain warranty authorization by calling your local Adapt Solutions dealer or Adapt Solutions directly at 1-866-641-0419.
2. Return the faulty Adapt Solutions component/equipment, freight prepaid, to the address provided by your Adapt Solutions dealer or Adapt Solutions with proof of purchase indicating the date purchased.

Adapt Solutions will pay for shipping back to the purchaser within the continental United States and Canada if a defect in materiel or workmanship is discovered. Return freight and repair charges will be the responsibility of the purchaser if the problem is not covered by warranty.

This warranty gives you specific legal rights, and you may also have other rights that vary from state to state/ province to province. Adapt Solutions specifically does not authorize any person to extend the time or scope of this warranty.

For further information regarding this limited warranty, please contact us by calling 1-866-641-0419 or by writing at the following address:

**Adapt Solutions/Warranty Department
145 Damase-Breton
St-Lambert-de-Lauzon, Québec
G0S 2W0 Canada**

WARRANTY REGISTRATION

Adapt Solutions #1 Ltd located in St-Lambert-de-Lauzon, Québec, Canada, warrants this product to be free from any manufacturer's defect and workmanship for a period of three years from the date of purchase. This product has to be installed by an authorized dealer and has to be operated properly to be warranted. This warranty is limited to the original purchaser and does not cover defects on the vehicle. The warranty registration must be completed and returned within 30 days of purchase. This warranty does not cover abnormal wear or damages resulting from an accident, misuse, neglect or if the product has been altered in any way or aspect. Defective parts must be returned by an authorized dealer, who will have received a prior return authorization from Adapt Solutions #1 Ltd. All warranted parts shall be sent to Adapt Solutions #1 Ltd prepaid. Parts will be replaced or repaired at Adapt Solutions #1 Ltd.

Date installed: _____

Dealer: _____

Dealer signature: _____

TO KEEP FOR YOUR RECORDS

(Place serial number here)

WE ARE THE SOLUTION

Please contact your dealer for immediate assistance

Dealer:

TF 1.866.641.0419 | tech@adaptsolutions.com



145 Damase-Breton | Saint-Lambert, Qc G0S 2W0

TF 1.866.641.0419 | **Fax** 418.889.9838 | **P** 418.889.0419

www.adaptsolutions.com

